- D** **Glaswaage**
Gebrauchsanweisung
 - GB** **Glass bathroom scale**
Instruction for use
 - F** **Pèse-personne en verre**
Mode d'emploi
 - E** **Báscula de vidrio para personas**
Manual de instrucciones
 - I** **Bilancia in vetro**
Istruzioni per l'uso
 - TR** **Cam terazı**
Kullanım kılavuzu
 - RUS** **Стеклянные весы**
Инструкция по применению
 - PL** **Waga szklana**
Instrukcja obsługi
 - NL** **Weegschaal met glazen weegplateau**
Gebruiksaanwijzing
 - P** **Balança de vidro**
Instruções de utilização
 - GR** **Γυάλινη ζυγαριά**
Οδηγίες χρήσης
 - DK** **Glasvægt**
Betjeningsvejledning
 - S** **Glasvåg**
Bruksanvisning
 - N** **Glassvekt**
Bruksveiledning
 - FIN** **Lasivaaka**
Käyttöohje
 - CZ** **Skleněná váha**
Návod k použití
 - SLO** **Steklena tehtnica**
Navodila za uporabo
 - H** **Üvegmérleg**
Használati utasítás
 - RO** **Cântar de sticlă**
Instrucțiuni de utilizare
- ميزان زجاجي
دليل الاستعمال

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- GB** Getting started
- F** Mise en service
- E** Puesta en marcha
- I** Messa in funzione
- TR** İlk çalıştırma

- D** Batterie einlegen oder Isolierschutzstreifen entfernen.
- GB** Insert batteries or remove insulating strip.
- F** Insérez les piles ou retirez la bande de protection isolante.
- E** Colocar las pilas o quitar las tiras de protección de aislamiento.

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GB General advice

- Max. capacity 180 kg (396 lb / 28 st), Graduation 100 g (0.2 lb / 1 lb).
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water.
- Protect the scale against knocks, damp, dust, chemicals, drastic changes in temperature, elec-

tromagnetic fields and sources of heat that are too near.

- Never step onto the outer edge of the scale with one foot as this could cause the scales to tip!
- Keep packaging material away from children!
- Not intended for commercial use.
- Should you have any questions regarding operation and usage of our appliances, please contact your local retailers or Customer Service.
- Before making a claim, please check the batteries and replace them if necessary.
- Repairs must only be carried out by Customer Services or authorised suppliers.
- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
- **⚠ Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.
- **⚠ Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.
- The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.
- The codes below are printed on batteries containing harmful substances:
Pb = Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury.
- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

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