DE	Klopf-Massagegerät Gebrauchsanweisung.....	2
EN	Tapping massager Instruction use.....	11
FR	Appareil de massage par tapotement Mode d'emploi	20
ES	Aparato de masaje por golpeteo Instrucciones para el uso	29
IT	Apparecchio per massaggi a impulsi Istruzioni per l'uso	38
TR	Tapotman masaj cihazı Kullanma Talimatı	47
RU	Прибор для похлопывающего массажа Инструкция по применению	56
PL	Urządzenie do masażu ostukującego Instrukcja obsługi	65

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

⚠ WARNING

- The unit is only intended for domestic/private use, not for commercial use.
- This device may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely, and are fully aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- The device has a hot surface. Persons insensitive to heat must be careful when using the device.
- Do not insert any objects into the opening in the device or into the moving parts. Ensure that the moving parts can move freely at all times.
- Do not force or jam any part of your body, in particular your fingers, between the moving massage elements or their holder in the device.
- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to comply will invalidate the warranty.

Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,

Your Beurer Team

Contents

1. Included in delivery.....	12	8. Operation.....	16
2. Explanation of symbols	12	9. Care and storage	17
3. Getting to know your appliance	13	10. In the event of problems?.....	18
4. Proper use	13	11. Technical data	18
5. Safety notes.....	14	12. Disposal.....	18
6. Appliance description.....	15	13. Warranty /service.....	19
7. Start-up	16		

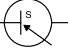
1. Included in delivery


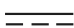
Check that the device packaging has not been tampered with and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

- 1 x Tapping massager
- 4 x Interchangeable massage attachments
- 1 x Mains part
- 1 x These operating instructions

2. Explanation of symbols

The following symbols are used in the operating instructions and on the type plate:

	WARNING Warning of risks of injury or health hazards.
	CAUTION Safety information about possible damage to appliance/accessories.
	Note Important information.
	The device has double protective insulation and therefore complies with protection class 2.
	Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).
	Dispose of packaging in an environmentally friendly manner
	Manufacturer
	CE labelling This product satisfies the requirements of the applicable European and national directives.
	Only use indoors
	Switching mains part
	Short-circuit-proof safety isolating transformer

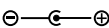	Polarity
	The products demonstrably meet the requirements of the Technical Regulations of the EAEU.
	Direct current The device is suitable for use with direct current only
	Energy efficiency level 6
IP20	Protected against solid foreign objects 12.5 mm in diameter and larger

3. Getting to know your appliance

Monitor functions

This electric massager allows you to give yourself or other people soothing massages. Massages can be relaxing or stimulating, and are popular for treatment of stiff or tense muscles, pains and fatigue. The device provides a powerful, intensive massage for the back, neck, shoulders, upper arms, thighs and calves. The device is for **private use only** and is not intended for medical or commercial purposes.

4. Proper use

This device is intended only for the massaging of individual parts of the human body. It cannot replace medical treatment. Do not use the massage device if any of the following warnings apply to you. Please consult your doctor if you are unsure whether or not the massage device is suitable for you.

WARNING

Use of the massage device is prohibited in the following cases:

- On animals.
- On persons suffering from a herniated disk or from abnormal changes to the vertebrae.
- Under any circumstances for massages in the area around the heart if you wear a pacemaker. If this is the case, consult your doctor regarding other parts of the body first.
- Never use the massage device on swollen, burned, inflamed or damaged skin or body parts.
- The massage device must not be used during pregnancy.
- Do not use the massage device on the face (eyes), on the larynx or on other particularly sensitive parts of the body.
- Never use the device while you are asleep.
- Never use the device after taking medication or consuming alcohol (reduced levels of awareness).
- On bones (e.g. joints, spine).
- In a vehicle.

You should consult your doctor before using the massage device if:

- you suffer from a serious illness or are recovering from an operation,
- you suffer from any of the following: common cold accompanied by a fever, varicose veins, thrombosis, phlebitis, jaundice, diabetes, nervous system diseases (e.g. sciatica) or acute inflammations.

The device is only intended for the purpose described in these operating instructions. The manufacturer cannot be held liable for damage occurring as a result of inappropriate or careless use.

5. Safety notes

Please read these operating instructions carefully. Failure to observe the information provided below can lead to personal injury or material damage. Keep these instructions in a safe place where they can also be read by other persons using the device. Always pass on these instructions together with the appliance if you sell it or pass it on.

WARNING

Keep children away from the packaging materials. Risk of suffocation.

Electric shock

WARNING

Just like any other electrical appliance, this massage device must be handled with due care and attention in order to prevent the risk of electric shock.

For this reason, the device must only be operated as follows:

- with the mains voltage specified on the device only,
- never use the device if any damage to the appliance itself or to any of its accessories is evident,
- never during a thunderstorm.

In the event of defects or operational faults, switch the device off immediately and, if applicable, disconnect it from the mains socket. Never pull on the power cable or on the appliance in order to disconnect the plug from the mains outlet. Never hold or carry the device by the power cable. Keep the cables away from hot surfaces. Never open the device for any reason.

Make sure that the massage device, the plug and the cable do not come into contact with water or other fluids.

Never try to retrieve the device if it has fallen into water and is connected to a socket through the mains adapter. Disconnect the mains adapter immediately.

Do not use the device if it or its accessories are visibly damaged.

Do not drop the device or subject it to any impacts.

For this reason, the device must be operated as follows:

- only in dry inside rooms,
- only with dry hands,
- never in the bath, in the shower or near a filled basin,
- never in swimming pools, whirlpools or in the sauna,
- never out in the open.

Never try to retrieve the device if it falls into water. Disconnect the plug immediately from the mains outlet.

Notes on handling rechargeable batteries

- If your skin or eyes come into contact with fluid from the battery cell, flush out the affected areas with plenty of water and seek medical assistance.
- Risk of explosion! Never throw batteries into a fire.
- Do not disassemble, split or crush the rechargeable batteries.
- Only use chargers specified in the instructions for use.
- Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.
- Fully charge the battery prior to initial use (see Chapter 7).
- In order to achieve as long a battery service life as possible, fully charge the battery at least twice per year.

Repairs

WARNING

Repairs to electrical appliances must only be carried out by qualified persons. Incorrect repairs can lead to considerable danger for the user. In the event of repairs, please contact our customer service or an authorised dealer.

Fire hazard

WARNING

There is a risk of fire if the device is used incorrectly or if it is not used according to these operating instructions.

For this reason, the massage device must only be operated as follows:

- Never operate the massage device for longer than 20 minutes (risk of overheating) and allow it to cool off for 20 minutes before using it again.
- Never leave the device unsupervised, particularly if children are around.
- Never use the device under covers, such as blankets or pillows.
- Never use near petrol or other highly flammable substances.

Handling


CAUTION

Switch the device off and disconnect the power supply after every use and before every cleaning.


- Make sure that all moving parts can move freely at all times. Wear a hair net if necessary.
- Make sure that no hair can get caught in the device.
- Avoid contact with sharp or pointed objects.
- Do not kink the cable.
- Make sure that the device is not too close to heat sources such as ovens, radiators or direct sunlight.
- Never insert objects into the device through any openings.
- Switch off the device and disconnect it from the mains when you are removing or attaching the handle or massage attachments.

6. Appliance description


Tapping massager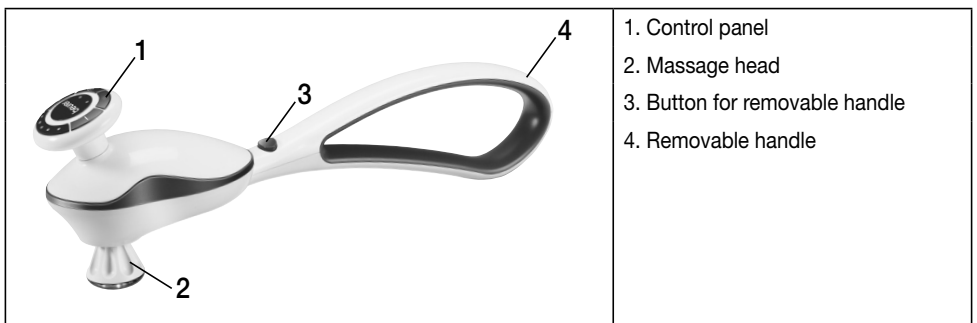


Exchangeable massage attachments

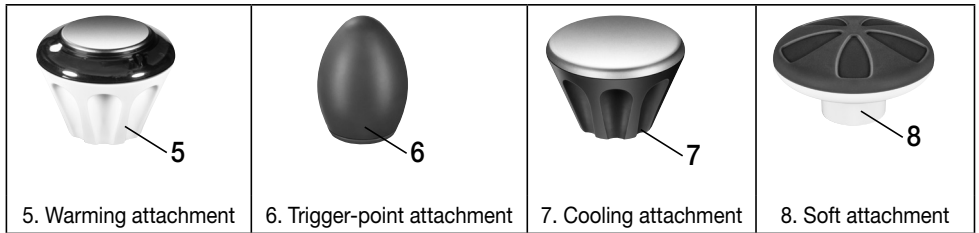Removable massage head

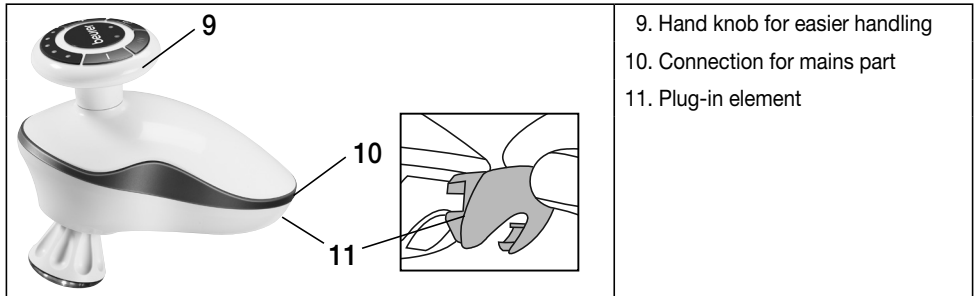Control panel

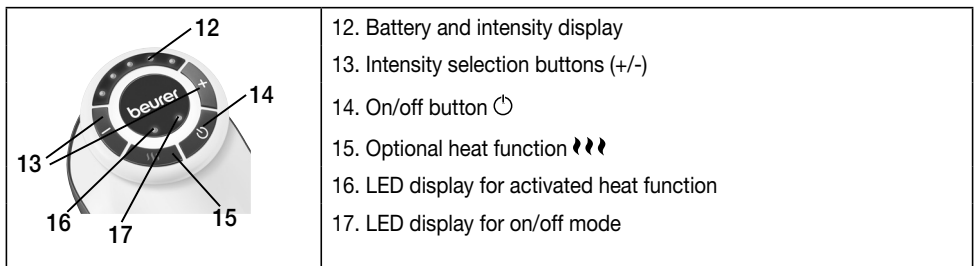7. Start-up

- Remove the packaging.
- Check the device, mains part and cable for damage.
- Insert the connector plug into the socket on the massager to charge the device.
- Charge the device for at least 3 hours before initial operation.
- When the device starts to charge, an acoustic signal sounds. During the charging process, the LEDs on the battery display flash from left to right, depending on the battery state.
- Once the device is fully charged, all 5 LEDs light up and an acoustic signal sounds 3 times.
- If the battery is low, the LED display for the on/off mode (green light) starts to flash and an acoustic signal sounds 3 times. If the battery is flat, the device switches off and an acoustic signal sounds again 3 times.
- The device can also be operated with a mains part. There is no charging in this case, however.
- Route the cable safely so that there is no risk of anyone tripping over it.

8. Operation

- The tapping massager can be used with or without the handle. It can be used with the handle to massage hard-to-reach parts of the body such as the shoulder and back area. It can be used without the handle for a more intensive, stronger massage.

- To remove the handle, please press and hold down the button (3) and at the same time pull the handle (4) away from the massage head. Then insert the plug-in element (11) in the back of the massage head to cover the handle attachment mechanism.
- Hold the massage head by the hand knob (9) so that you can guide the massage head securely.
- If you want to reattach the handle (4), first pull out the plug-in element (11) and push the handle (4) back on until you feel and hear it engage.
- You can use various massage attachments for massaging. To do this, select one of the 4 interchangeable massage attachments of your choice.
- Screw the attachments onto the massage head in a clockwise direction. To replace or remove the attachments, turn these anti-clockwise.
- Switch on the massager with the on/off button (⏻).
- When you press the buttons, a short acoustic signal sounds as confirmation. After this, the current battery state is displayed for 2 seconds and the massage then starts with the intensity level that was most recently selected.
- For the warming attachment (5), the application of heat can also be switched on or off by pressing the 🔥 button once. Press 🔥 again to switch off the heat function. The heat function is only available with the supplied warming attachment (5).
- Use the intensity selection buttons (+/-) to set the massage intensity you want. There are 5 different massage intensities available. Press the (+) button for a higher intensity. Press the (-) button for a lower intensity.
- Move the device to the part of the body you want to massage. You may be sitting or standing for this and you can use the device to massage yourself or another person.
- You can change the massage pressure by pressing the massager firmly or more gently.
- After a maximum usage time of 20 minutes, switch the device off and leave it to cool down.
- The device also has an automatic switch-off time of 20 minutes.
- Keep the device dry.

Note

Do not use the massage device just before going to bed. The massage can also have a stimulating effect and may cause difficulties in getting off to sleep.

WARNING

- The massage must be pleasant and relaxing at all times. Stop the massage or change the position of the massage device if you find it is painful or unpleasant.
- Do not jam or force any part of your body, in particular your fingers, into the area of the massage head.

9. Care and storage

Cleaning

First unplug the device and allow it to cool off. Then you can clean it with a cloth which is no more than lightly moistened. You can also use a mild detergent.

WARNING

- Disconnect the device before cleaning it.
- Only clean the appliance in the manner specified. Fluid must never be allowed to enter into the device or accessories.
- Do not use the device again until it is completely dry.

Storage

Keep the massage device in a cool and dry place where it is out of the reach of children. We recommend storing the device in its original packaging.

10. In the event of problems?

Problem	Cause	Remedy
Massage head rotates too slowly.	The load on the massage head is too great.	Free up massage head.
Massage head does not move.	Battery power is too low or device is not connected to mains.	Charge the device.
	Device is not switched on.	Switch on the device.
	Device has switched off automatically (automatic switch-off).	Stop use and allow the device to cool down.
Heat function will not switch on.	The warming attachment is not on the massage head.	Attach the warming attachment.

11. Technical data


Mains operation:	
Input	100-240 V~, 50/60 Hz, 0,8 A
Output	9,0 V === 2,5 A, 22,5 W
Average active efficiency	≥ 88,68 %
Efficiency at low load (10 %)	≥ 87,88 %
No-load power consumption	≤ 0,044 W
Dimensions	45.5 x 10 x 15.5 cm
Weight	approx. 900 g
Battery:	
Capacity	2600 mAh
Nominal voltage	7.4 V
Type designation	Lithium-ion

12. Disposal

CAUTION

- The empty, completely flat rechargeable batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the rechargeable batteries.
- Remove the battery from the device by carefully opening the massager. To do this, unscrew all of the screws on the housing with a screwdriver and remove the housing cover. Then take the battery out of the device and dispose of it correctly.
- The codes below are printed on rechargeable batteries containing harmful substances:
Pb = Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury.

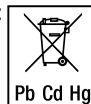For environmental reasons, do not dispose of the device in the household waste at the end of its useful life.

Dispose of the device at a suitable local collection or recycling point.

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

The device battery must be disposed of separately. The battery must not be disposed of with domestic waste.

13. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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